



Don/Doff Protocol for Repetitive Use of N95

N95 DONNING

1. Gather Supplies- Container labelled with name, gloves, 2 pieces of paper towel, disinfectant wipes or spray.
2. Sanitize hands
3. Don gloves
4. Remove and discard surgical mask
5. Remove the lid from your container and place upside down on a paper towel.
6. Hold the container directly in front of your face. Grab the outside of the container and place palm of hand underneath straps.

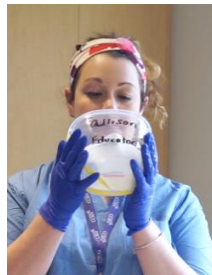

7. Align bridge of nose with the top of your N95, identify the top strap, and slide the top strap of your N95 to the top of your head.

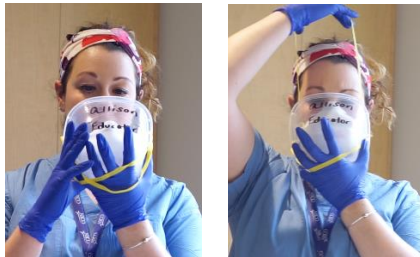

8. Slide the second strap over your head. Before adjusting the fit of your N95 place container on paper towel. Sanitize hands. Adjust fit of N95.



7. Disinfect inside of container, remove gloves, and sanitize hands
8. Proceed with final steps of Donning PPE.

Don/Doff Protocol for Repetitive Use of N95

N95 DOFFING

1. Sanitize hands
2. Don gloves
3. Remove lid and place on paper towel
4. Hold the container in front of your face.
5. Remove the bottom strap of the N95 mask and slide the strap over your head and onto the back of the container.



6. Take the upper strap from the N95 and slide this strap over your head, and snap onto the back of the container.
 - a. *Your container should now look like this:*



7. Place container on paper towel. Snap lid onto the container, disinfect the outside and the top of the container with a germicide. Remove gloves and sanitize hands.
8. Store your N95 in the container in a designated location per your unit.
9. At the end of your shift, sanitize hands, don gloves, remove your N 95 from the container, discard N 95 into trash, disinfect the inside and the outside of the container with germicide, remove gloves and sanitize hands, and store the container for reuse.